EN - ENGLISH

CUANTUM Light Curing Oral Adhesive Oral tissue adhesive

Important: Please read.

Disclaimer:

This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Quantum Medical Cosmetics s.l. (CMC). Non-recommended use of products made by third parties in conjunction with CMC products will void any warranty or other obligation, express or implied, of CMC. The user of CMC products has the duty to determine whether or not any product is suitable for the particular patient and circumstances. CMC disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of CMC products. The user is also obliged to study the latest developments in regard to this CMC product and its applications regularly. In cases of doubt, the user has to contact CMC. Since the utilization of this product is under the control of the user, it is his/her responsibility. CMC does not assume any liability whatsoever for damage arising thereof. Please note that some products detailed in this Instructions For Use may not be regulatory cleared, released or licensed for sale in all markets.

Description:

Quantum Light Curing Oral Adhesive is a medium viscosity cyanoacrylate adhesive composed of n-butyl cyanoacrylate. It is provided in single-dose 1 g aluminium tube in a sealed pouch (multipack of 10 units) that is co-packed with 1 blue-visible LED lamp and 10 disposable precision dispensing tips. Quantum Light Curing Oral Adhesive remains liquid until it is applied to the target site and irradiated with the blue-visible LED light during around 10-20 seconds at a distance of ca. 2 cm, forming a solid flexible film that bonds to the underlying surfaces.

Indications:

Quantum Light Curing Oral Adhesive is intended to protect and seal affected areas in oral wounds and incisions, as unique sealant or in conjunction with regular sutures. The formed film upon adhesive solidification acts as a wound closure system and mechanical barrier that retains the clot blood and avoids contamination by foreign matter.

Contraindications:

Quantum Light Curing Oral Adhesive is not to be applied below the surface of the skin. Quantum Light Curing Oral Adhesive is not to be used on patients with known preoperative systemic infections, uncontrolled diabetes, or diseases or conditions that are known to interfere with the wound healing processes.

Warnings:

- Do not apply in patients with allergy to cyanoacrylates.
- In rare cases this material may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Cautions:

Storage and handling:

- The product needs to be stored in mild conditions of temperature and humidity and out of direct light. Do not use after the use by date indicated on the package.
- The adhesive is capable of adhering to most body tissues (adheres skin and eyelids within seconds) and many other materials, such as latex gloves and stainless steel. Avoid contact with any tissue or equipment that is not disposable or that cannot be readily cleaned with a solvent, such as acetone. Do not use acetone or other chemicals to remove the product from undesired locations inside the patient's mouth, or other skin.
- If accidental contact with the eye occurs, do not attempt to open the eye. Cover the eye with a soft dressing and consult an ophthalmologist immediately. The adhesive will lose its adhesive properties in a few days, and the eye will open spontaneously with no damage.

- The product is not absorbable and should only be used as a surface device. Do not allow it to ingress beneath surface tissues.

- A small amount of heat is generated during the process of polymerization, however, this will only last a few seconds and cause the patient no discomfort.

- Do not use in the presence of gauze, cotton tipped applicators, basic substances or polyurethane foam applicators. These materials may cause excessive exothermic (heat generating) reaction.

- Quantum Light Curing Oral Adhesive will not adhere to surfaces pre-coated with petroleum, therefore avoid using this substance on any area where the product is intended to adhere.

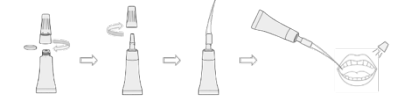
- Do not apply any type of cream or lotion on the surface intended to be treated before using Quantum Light Curing Oral Adhesive.

Intended users and intended patients:

- This product is for sale and use only by dental medical specialists (professional use only) in patients undergoing dental surgery.
- It is recommended that any person intending to use Quantum Light Curing Oral Adhesive has the necessary experience and tested the material outside of a patient to get used to the proper handling of the product.

- Keep out of reach of children.
- Directions for use / Handling procedure:** Previous to application: Clean the affected area with water and dry. Application:

- Open the box containing the products.
- Peel open the protective aluminium pouch and remove the aluminium tube containing the adhesive.
- Unscrew the tip-cap assembly, remove the secure ring, and insert back the tip-cap assembly by screwing it in the aluminium tube.
- Open the plastic bag and remove a precision dispensing tip.
- Remove the cap from the aluminium tube and insert the precision dispensing tip by pressing down.
- For the following steps take care not to touch the adhesive with anything not intended for this purpose besides the patient's treatment area.
- Make sure to clean the wound so that no blood excess is present on the sites intended to be treated with Quantum Light Curing Oral Adhesive.
- Approximate the wound edges (if applicable) either manually or using the suturing technique you deem applicable to ensure uneventful wound healing.
- Apply over the target site in a thin layer and expose it to the blue-visible light.

- The adhesive will set in 10-20 seconds (the adhesive sets faster when applied in small amounts and spread thinly).
- A second layer can be applied and polymerized over the first one if it is considered convenient.
- Make sure the adhesive is totally cured before allowing it to come in contact with other areas of patient's mouth.
- Once finished with the application, safely discard the aluminium tube and the precision dispensing tip.

Materials:

Quantum Light Adhesive contains 10 units of 1 g of product in an aluminium tube inside a polyfoil sachet. It is co-packed with 1 blue-visible LED lamp and 10 disposable precision dispensing tips for the clinical application.

Cleaning instructions:

The liquid adhesive can be easily removed with the aid of a cellulose tissue. Tools that are contaminated with solidified Quantum Light Curing Oral Adhesive may be cleaned with acetone.

Caution: please make sure that the tools can withstand this treatment without damage (i.e. some plastics such as PVC and polystyrene can be dissolved by acetone) and clean the instruments with repeated rinsing in water and subsequent autoclaving if necessary according to the manufacturer recommendations.

Disposal:

The packaging of Quantum Light Curing Oral Adhesive can be disposed of in normal office waste.

Symbols and Labels:

Symbol	Title of Symbol
	Catalogue number Lot number
	Manufacturer
	Date of manufacture
	Do not re-use
	Use-by date
	Keep dry
	Keep away from sunlight
	Consult instructions for use
	For prescription use only

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