EN - ENGLISH

CUANTUM MediGlue
High viscosity topical tissue adhesive
Important: Please read.

Disclaimer:

This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Cuantum Medical Cosmetics s.l. (CMC). Non-recommended use of products made by third parties in conjunction with CMC products will void any warranty or other obligation, express or implied, of CMC. The user of CMC products has the duty to determine whether or not any product is suitable for the particular patient and circumstances. CMC disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of CMC products. The user is also obliged to study the latest developments in regard to this CMC product and its applications regularly. In cases of doubt, the user has to contact CMC. Since the utilization of this product is under the control of the user, it is his/her responsibility. CMC does not assume any liability whatsoever for damage arising thereof. Please note that some products detailed in this Instructions For Use may not be regularly cleared, released or licensed for sale in all markets.

Description:

Cuantum MediGlue is a high viscosity cyanoacrylate adhesive composed of n-butyl cyanoacrylate. Cuantum MediGlue is provided in a mono-dose folded aluminium tube in aluminium sealed pouch.

Cuantum MediGlue remains liquid until it is applied to the target site. Upon contact with moisture and proteins found on surface areas, Cuantum MediGlue cures to form a film that bonds to the underlying surfaces.

Indications:

Cuantum MediGlue is a topical tissue adhesive intended to hold closed easily approximated edges of low-tension wounds from clean surgical incisions and small, thoroughly cleansed, trauma-induced lacerations. Cuantum MediGlue may be used in conjunction with, but not in place of deep dermal sutures.

Contraindications:

Cuantum MediGlue is not to be applied below the surface of the skin. Cuantum MediGlue is not to be applied to wounds subject to high skin tension.

Cuantum MediGlue is not to be used on patients with known preoperative systemic infections, uncontrolled diabetes, or diseases or conditions that are known to interfere with the wound healing processes.

Warnings:

- Do not apply in patients with allergy to cyanoacrylates.
- In rare cases this material may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Cautions:

Storage and handling:

- The product needs to be stored at room temperature in a dry place and out of direct light. Do not use after the use by date indicated on the package.
- The adhesive is capable of adhering to most body tissues (adheres skin and eyelids within seconds) and many other materials, such as latex gloves and stainless steel. Avoid contact with any tissue or equipment that is not disposable or that cannot be readily cleaned with a solvent, such as acetone. Do not use acetone or other chemicals to remove the product from undesired locations on the patient's skin.
- If accidental contact with the eye occurs, do not attempt to open the eye. Cover the eye with a soft dressing and consult an ophthalmologist immediately. The adhesive will lose its adhesive properties in a few days, and the eye will open spontaneously with no damage.
- The product is not absorbable and should only be used as a surface device. Do not allow it to ingress beneath surface tissues.
- A small amount of heat is generated during the process of polymerization. Apply thinly to make sure that no discomfort is caused to the patient.

- Do not use in the presence of gauze, cotton tipped applicators, basic substances or polyurethane foam applicators. These materials may cause excessive exothermic (heat generating) reaction.

- Cuantum MediGlue will not adhere to surfaces pre-coated with petrolatum, therefore avoid using this substance on any area where the product is intended to adhere.

- Do not apply any type of cream or lotion on the surface intended to be treated before or after using Cuantum MediGlue.

Intended users and intended patients:

- This product is for sale and to be used on low tension wounds.


- Keep out of reach of children.

Directions for use / Handling procedure:

Previous to application: Clean the affected area with water and dry.

Application:

- 1) Peel open the protective pouch and remove the tube and the cap-tip assembly. Discard the pouch in regular office waste.

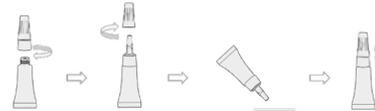- 2) Screw the cap-tip assembly clockwise to perforate the aluminium foil.
- 3) Remove the cap unscrewing counter clockwise.
- 4) Approximate the wound edges and dispense a small amount of adhesive over them by pushing the aluminium tube.
- 5) Spread the adhesive in a thin layer with the aid of the applicator tip, moving it horizontally to completely cover the wound edges (thick layers will increase the setting time and could sloughs of prematurely).
- 6) Let the adhesive dry before releasing the wound edges.
- 7) Once finished the application, close the aluminium tube by screwing the cap clockwise again and discard the device in the general waste.

Materials:

Cuantum MediGlue multipacks contain 12 individual pouches, each containing 1 folded aluminium tube and 1 cap-tip assembly.

Ingredients:

n-Butyl cyanoacrylate, glyceryl triacetate, vinyl chloride/vinyl acetate copolymer, fumed silica, D&C violet 2.

Cleaning instructions:

The liquid adhesive can be easily removed with the aid of a cellulose tissue. Tools that are contaminated with solidified Cuantum MediGlue may be cleaned with acetone.

Caution: please make sure that the tools can withstand this treatment without damage (i.e. some plastics such as PVC and polystyrene can be dissolved by acetone) and clean the instruments with repeated rinsing in water and subsequent autoclaving according to the manufacturer's recommendations.

Disposal:

The product does not require specific waste treatment; it can be disposed of in normal office waste.

Symbols and Labels:

Symbol	Title of Symbol
	Catalogue number
	Lot number
	Manufacturer
	Date of manufacture
	Do not re-use
	Use-by date
	Keep dry
	Keep away from sunlight
	Consult instructions for use

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All rights reserved, Quantum Medical Cosmetics s.l.