


**CUANTUM⁺**
LIQUID BANDAGE

SEALS AND PROTECTS

- Cuts
- Scratches
- Chafings
- Hangnails

Transparent
Waterproof
Fast Drying
50 applications



NET CONTENT
1.5 g e


**CUANTUM⁺**
LIQUID BANDAGE

DIRECTIONS

Preparation:

Clean the affected area with water and dry.

Use instructions:

- 1) Remove the cap and dispense a small drop of product over the target site.
- 2) Spread it uniformly forming a thin layer.
- 3) The adhesive will solidify in 30-60 seconds creating a protective film that sloughs off naturally as healing occurs.
- 4) After the application clean the applicator tip with a tissue and screw the cap back until the following use.

Storage:

Keep at room temperature in a dry place, avoiding direct sun.

WARNING

- Only for external use.
- Do not use in or around the eyes or on burns, wounds or skin with active signs of infection.
- In case of eye contact do not try to open the eye. Cover it with a soft dressing and consult an ophthalmologist immediately. The adhesive will lose its adhesive properties in a few days, and the eye will open spontaneously with no damage.
- Contains cyanoacrylate. Bods skin and eyelids within few seconds.
- Do not use if allergic to cyanoacrylate.
- Keep out of reach of children.

 Quantum Medical Cosmetics S.L.,
Edifici Eureka, Campus UAB S/N, Cerdanyola del Vallés
Barcelona, Spain, +34 658 875 835 - www.cuantum.me

Medical Device
Rev. 08/2019

Distributed by:

